The following products (code Nos.) are used in our application examples:


٠	Pre-po	re-polishing of large areas using pumice						
	0	Circular brush – goat hair Ø 90 mm				32/90 mm	(SRB 12 100)	
	 Circular brush with Chungking bristles Particularly beneficial are the LABOR and PRIMA brushes with several rows of bristles, converging type, and also the straight type with three rows. (Please choose your favorite brush from our catalogue, pages 37 to 41). Pre-polishing of interdental areas using pumice 							
•								
		Lathe brush, Lathe brush,	sh, Suprem goat hair v goat hair v	a, 1 row a, calico interleaved with aluminum center Ø 50 mm with aluminum center Ø 60 mm with plastic center Ø 48 mm		1/25/50 SH 3/40/80 SKLs 152/50 152/60 162/48	(RSH 11 100) (RST 34 100) (SBA 12 100) (SBA 12 200) (SBK 12 100)	
٠	Pre-polishing a deep palatal area using pumice							
	0	Felt cones, particularly beneficial with Ø 20 mm and Ø favorite felt cone from our catalogue on page 34) Dental plate brush					oose your (KOB 12 100	
•	High-gloss polishing							
	 Buffing wheel, cotton cloth, particularly beneficial: Pleated buffing wheel 				29a/100KK fine 29a/100KK coarse 29d/100KK		(SWK 22 200) (SWK 23 200) (SWK 28 100)	
•	Repolishing of difficult places							
	 Miniature brushes, goat hair, white, mounted 							
		Ø 19 mm Ø 21 mm Ø 23 mm Ø 25 mm	112/19 112/21 112/23 112/25	(MBH 12 200) (MBH 12 300) (MBH 12 400) (MBH 12 500)				
	 Miniature brushes, goat hair, white, unmounted 							
		Ø 19 mm	122/19	(MBU 12 200)				

Polishing Acrylic Dentures

We recommend the following articles in our range of products. In this information brochure we mainly focus on the particularly tricky places for polishing.

> Circular brush with Chungking bristles



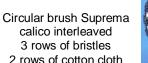



Circular brush Suprema,

calico interleaved 3 rows of bristles 2 rows of cotton cloth



1 row







Pleated buffing whee

Buffing wheel





for pre-polishing







Tel: 07731-947 220

Josef-Schüttler-Str. 49 78224 Singen, Germany Fax: 07731-947 221

129N/23 (MBU 22 400) 129N/25 (MBU 22 500) Wishing you good polishing results

(MBU 12 300)

(MBU 12 400)

(MBU 12 500)

(MBH 22 200)

(MBH 22 300)

Miniature brushes with discs of cotton cloth, unmounted

• Miniature brushes with discs of cotton cloth, mounted

102N/23 (MBH 22 400) 102N/25 (MBH 22 500)

129N/19 (MBU 22 200)

129N/21 (MBU 22 300)

für die Politur Dr. Montemerlo GmbH & Co KG

Ø 21 mm 122/21

122/23

122/25

102N/19

102N/21

Ø 23 mm

Ø 25 mm

Ø 19 mm

Ø 21 mm

Ø 23 mm

Ø 25 mm

Ø 19 mm

Ø 21 mm

Ø 23 mm

Ø 25 mm

0





Circular brush with goat hair





Lathe brush with white goat hair



Miniature brushes with white goat hair







Blue polishing paste

Pre-polishing large surfaces with pumice

Both the Circular Brush with goat hair 32/90 mm (maximum speed of 4,000 rpm) and the Circular Chungking Brushes LABOR and PRIMA (recommended speed of 3.000 rpm) are suitable for pre-polishing larger surfaces, e.g. flat palatal areas.



Example:4/40/80 Prima KK converging type

Pre-polishing interdental spaces with pumice

Were you disappointed after the polishing because the denture teeth suffered badly during the final process? Those days are over! Interdental spaces can be polished without the denture teeth being polished "flat" afterwards.

The 1-row Circular Brush Suprema is well-suited for polishing interdental spaces when using a contact pressure that is not too high and a speed of approx. 3,000 rpm. Due to the arrangement of the bristles, the heat generation can be easily controlled despite their hardness.

Example:1/25/50 Suprema KK (RSK 11 100)

The Circular Brush Suprema with calico interleaved (3 rows of bristles and 2 layers of cotton cloth) has two big advantages:

- 1. One row of the Chungking bristles polishes the interdental space while the adjacent teeth are being protected by the interleaved cotton cloth.
- 2. The cotton cloth serves for absorbing moisture or polishing paste.

Speed of 3,000 rpm

Example :3/40/80 Suprema / calico (RST 34 100)

The Lathe Brush with white goat hair is suitable for polishing interdental spaces. However, a low contact pressure should be ensured and the maximum speed of 4,000 rpm should not be exceeded.



Example: Goat hair 162/48 mm (SBK 12 100)

Pre-polishing a deep palatal area with pumice

Have you ever been annoyed because the gums are still not polished, whereas the prosthesis in the area of the palatal vibrating line has already lost some of its substance? If your answer is "Yes", then it is time you used our Felt Cones and our Dental Plate Brushes. The recommended speed is 1,500 rpm for both.



Example:Felt cone 50/50x25 mm (FKU 25 200)

High-gloss polishing

A beautiful, high-gloss finish is achieved using our Buffing Wheel with cotton cloth together with the blue polishing paste (P3) (maximum speed of 4,000 rpm). Before using a new buffing wheel for the first time, the buffing wheel should be 'ground' slightly using a sharp object. As an alternative to the above buffing wheel with cotton cloth, we can also recommend the **Pleated Buffing Wheel**. The advantages are:

- 3. a better cooling effect
- 4. less fluffing

If used for pre-polishing, the buffing wheel with cotton cloth can also be applied together with wet pumice. In our range of products you will find an extra-stiff buffing wheel (31/100 mm) which is particularly well-suited for pre-polishing. The pleated buffing wheel should not be used with wet pumice, as it cannot be completely cleaned again due to the pleats.

Do you also find a place, now and again, that has not been polished so well? In this case we recommend our Miniature Brushes with goat hair or discs of cotton cloth which can be used both with pumice and with blue polishing paste at a maximum speed of 12,000 rpm.






Example: Dental plate brush No. 145 (KOB 12



Pleated buffing wheel 29d/100 mm (SWK 28

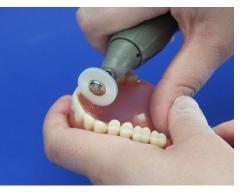

Example: Goat hair 112/21 mm (MBH 12 300)